


A U D E A R A

Perfect sound, always.

A U D E A R A

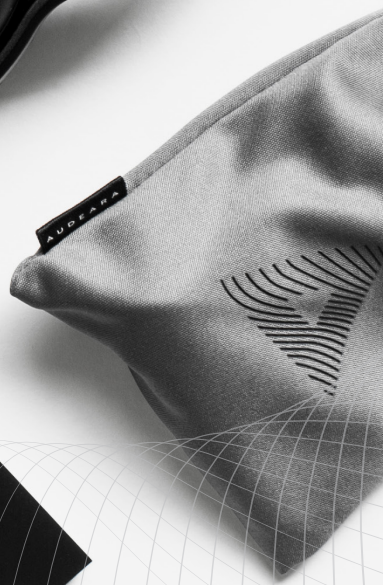

HOW TO GUIDE - LITE  
A-01 TV Bundle

# FEATURES

## Personalise your sound

Personalise your headphones and track your hearing over time. Experience crystal clear sound and quality while watching television, listening to music and communicating (telephone calls or video chat).

The Audeara headphones are compatible for use with cochlear implants and hearing aids and can be personalised via a simple hearing check using the Audeara app or at your hearing clinic.



### Phone Calls and Video Chat

Answer calls or connect to video chat on your headphones



### Connect to your TV

Watch your favourite TV shows or movies using the BT-01 transceiver



### Wireless

Seamless connection to any Bluetooth device including laptop, tablet or mobile phone



### Extra Long Battery Life

35+ hours with Bluetooth and noise cancellation



### Active Noise Cancellation (ANC)

Reduce unwanted background noise and turn on ANC, on the right ear cup of your headphones



### Multifunction buttons

Your headphones can answer calls, adjust volume or pause music



### Wireless Inbuilt Microphone

Perfect crystal clear sound in and out



### Connect 2 headphones

The transceiver can connect 2 Audeara headphones at the same time





## PAIRING THE A-01 HEADPHONES TO YOUR PHONE

1. Turn on the headphones using the power switch on the left ear cup.
2. Hold the middle button on the left ear cup of the headphones until the light flashes red and blue.
3. Open your phone and navigate to settings> Bluetooth and make sure it is on.
4. "Audeara A-01" will appear in your available Bluetooth devices.
5. Select "Audeara A-01" to connect.


## PAIRING THE BT-01 TV TRANSCEIVER



1. Plug BT-01 into a power source using the USB cable.
2. On the back of the BT-01, slide the button to "TX".
3. Turn on the headphones using the power switch on the left ear cup.
4. Hold the middle button on the left ear cup of the headphones until the light flashes red and blue.
5. On the BT-01, slide the button on the top to the left, and hold until it flashes red and blue.
6. The light on the BT-01 will turn a steady blue once paired.

# PLUGGING INTO YOUR TV

1. Look for the “Digital/Optical Audio Out” port at the back of your TV. If your TV does not have one, skip to step 3.
2. Plug one end of the Optical Cable into the “Digital/Optical Audio Out” port of the TV and the other end into the “Optical Audio In” at the back of the BT-01.
3. If you don’t have a “Digital/Optical Audio Out” port on your TV, plug the red and white adapter into your TV.
4. Plug the audio cable into the black end of the red and white adapter.
4. Plug the other end of the audio cable into the “Audio In” port on the BT-01.
5. On the top of the BT-01, slide the button to “OPT” if you are using the optical cable or “AUX” if you are using the red and white adapter and audio cable.

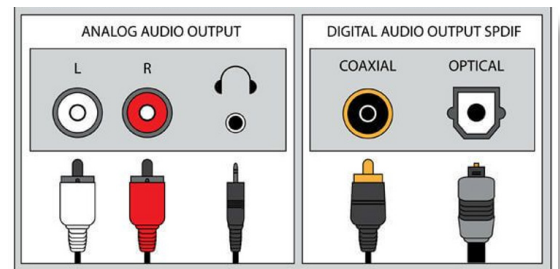

Audio (AUX) cable

USB charger



Red and white adapter

Optical cable

A U D E A R A

## CONTACT

support@audeara.com

1300 251 539